


PressureGuard[®] APM²

Alternating Pressure/Lateral Rotation with Deluxe Control Unit

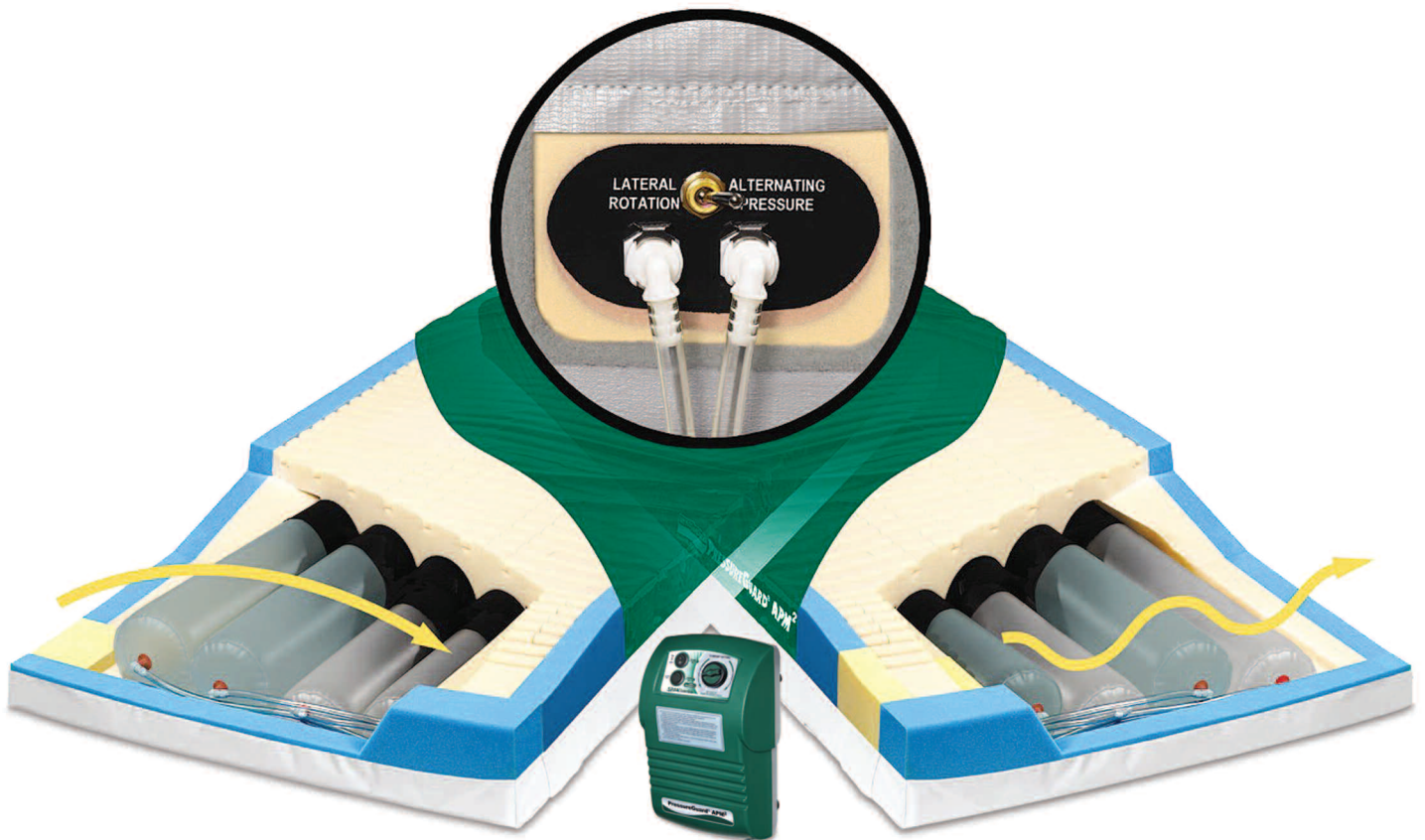

Twice the Flexibility

Choice of two therapies at the flip of a switch delivers double the value for busy caregivers.

Twice the Simplicity

Ease of operation and cleaning make it equally user-friendly in facility or at home.

Twice the Safety

Minimizes risks associated with edge-of-bed/entrapment issues and patient transfers.

SPANAmerica
Innovative Solutions.




PressureGuard[®] APM² with deluxe control unit

“Deuce Deluxe”: Two Therapies—at the flip of a switch.


The APM² allows caregivers to change instantly between two proven modalities by flipping an easy access toggle switch. The user experiences subtle but effective alternating pressure or lateral rotation in ten minute cycles. These dual therapies combine with the patented PressureGuard air/foam design to deliver outstanding tissue load management in any care setting.

Safety and Skin Integrity.

Exclusive PressureGuard[®] design delivers both!

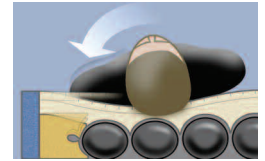


- **Patented “Safety Edge”** ensures safer transfers while minimizing the risk of falls and patient entrapment.
- **Patented “Air/Foam” integration** exceeds pressure management performance of air-only systems.

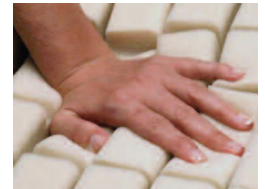

- **Patented Geo-Matt[®] Design** creates unique, anti-shearing surface with 800 individually responsive cells.



- **Patented Heel Slope[®]** redistributes load to calves while providing complete foot support.

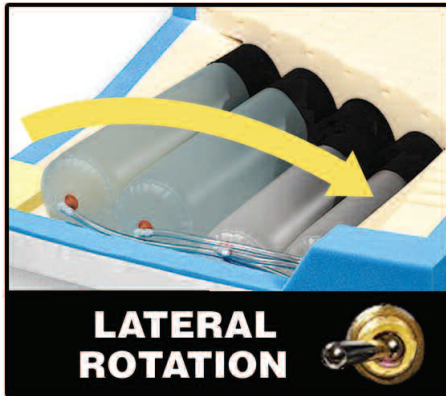

LATERAL ROTATION

Continuous Lateral Rotation Therapy (CLRT):


Beyond pure pressure management, CLRT gives caregivers a versatile tool that is indicated for a wide variety of patient needs¹, including:

- As a preventive measure against further complications associated with critically ill or immobilized patients
- To allow repositioning for conditions where pain makes manual repositioning difficult
- To decrease probability of fractures caused by manually repositioning patients with osteoporosis

Deluxe Control Unit:

Silent, reliable, maintenance-free control unit allows simple adjustment for user comfort.

- **Low Pressure indicator light** gives at-a-glance assurance of proper function
- **Selectable audible alarm** can be engaged to alert caregiver to low pressure status
- **Torque-free port** location helps eliminate mechanical stresses on quick-connectors, minimizing wear
- **Hospital-grade plug** with 15-foot cord
- **Heavy-duty hooks** ensure stability

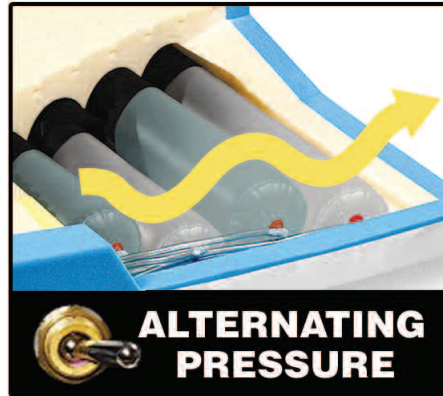

ALTERNATING PRESSURE

Alternating Pressure Therapy:

Typical alternating pressure mattresses create extreme pressure gradients between inflated and deflated cylinders. But the APM² does not, thanks to:

- An integrated foam shell, which surrounds and encapsulates the air system below, eliminating sharp pressure drop-offs
- Its unique longitudinal air system configuration, which allows greater air displacement and documented performance advantages² over typical side-to-side designs

- 1 For conditions requiring customizable rotation pattern, roll angle, or dwell time, the PressureGuard Turn Select[®] is indicated
- 2 Batt, Todd *Comparing Lateral and Longitudinal Air Cylinder Configuration in Support Surfaces*. Available on request

Item #	Description	Dimensions
5875LR-29	APM ² with deluxe control unit	75”L x 35”W x 7”H
5880LR-29	APM ² with deluxe control unit	80”L x 35”W x 7”H
5884LR-29	APM ² with deluxe control unit	84”L x 35”W x 7”H
5800	Deluxe control unit only	
Cover: Fluid-proof, coated nylon. Anti-fungal, bacteriostatically treated.		
Control Unit:		
Weight:	6.3 lbs.	Dimensions: 12” x 7.5” x 5”
Voltage:	120 AC	Max current: 1.0 Amp
Frequency:	60 Hertz	UL Listing: Certified to IEC 60601-1-1
Weight limit: 350 lbs		
Mattress weight: approx. 20 lbs.		
18-month warranty (all components)		

All models are fire barrier version (FBV) as indicated by “-29” suffix, denoting certified conformance to NFPA 101 (Life Safety Code); Cal Tech Bulletin #117; 16 CFR Parts 1632 and 1633; and BFD IX-11.



SPANAmerica
Innovative Solutions.